


Audiologist Ear Impression Guide

A full Impression is required for each ear. It must include the full helix, crus of the helix, tragus and antitragus.

IMPORTANT: The impression must be taken completely past the second bend of the ear canal.

Please overfill the outer ear including cymbal, crus of helix and surrounding areas. The impression material should be 'compressed' at the superior junction point of the Tragus.

(Be sure to examine the ear canal for any obtrusions or inflammation)

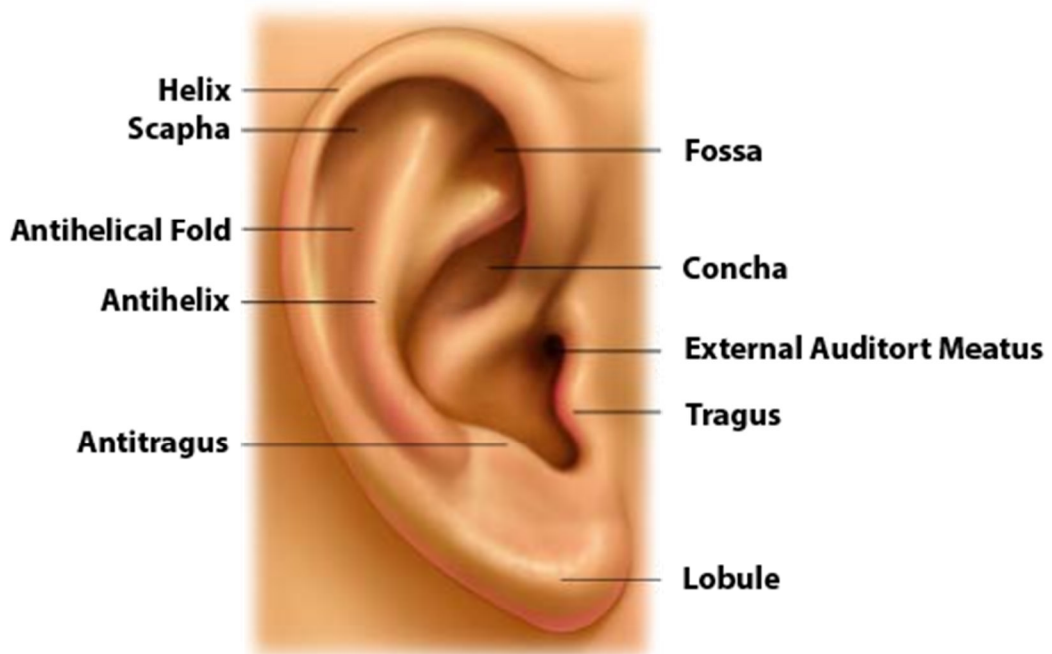

The impression should be made using a silicone base high viscosity impression material.

The impression should be taken with an 'open mouth position'. Use a bite block during the impression process to reduce jaw movement. This will give a better seal in the final product.

**IMPRESSIONS TAKEN WITH A LIQUID POWDER MIX
WILL NOT BE ACCEPTED**